

## **Instructions for Application of EAZI-BREED™ CIDR®:**

1. Wear gloves whenever handling CIDR®s.
2. Wash the CIDR® gun with disinfectant before and between each use.
3. Place the CIDR® insert, tail first, into the gun. The two wings will be pushed together and should be left about one inch above the top of the gun.
4. Apply generous amounts of lube to the tip of the CIDR® insert and gun.
5. Clean the vulva.
6. When inserting the gun and insert, make sure the tail of the CIDR® is on the underside of the gun and is curling down.
7. Insert the gun at a slightly upward angle, moving forward over the pelvic bone until it meets resistance.
8. Depressing the plunger on the gun will insert the CIDR® and then slowly remove the gun.
9. Clip the tail of the CIDR® to about 2 ½ inches (62.5 mm) from the vulva to prevent removal by curious penmates.
10. Remove the insert 7 days later by gently pulling on the tail of the CIDR®.

## **Instrucciones para la Aplicación del EAZI-BREED™ CIDR®:**

1. Use guantes cuando maneje los CIDR®.
2. Lave la pistola del CIDR® con desinfectante antes de cada uso, y entre cada uso.
3. Coloque la inserción del CIDR®, la parte de atrás primero, en la pistola. Las dos "alas" deben ser empujadas juntas, y deben dejarse alrededor de una pulgada por encima de la parte de arriba de la pistola.
4. Aplique una cantidad generosa de lubricante en la punta de la inserción del CIDR® y en la pistola.
5. Limpie la vulva.

6. Cuando inserte la pistola con el producto, asegúrese que la punta posterior del CIDR® está en la parte de debajo de la pistola, y arrollada hacia abajo.
7. Inserte la pistola en un ángulo un poco hacia arriba, moviéndola hacia delante sobre el hueso pélvico, hasta que encuentre resistencia.
8. Bajando el disparador de la pistola hará que el CIDR® se inserte; luego remueva la pistola con cuidado.
9. Corte la punta del CIDR® hasta unas 2 ½ pulgadas (6.25 cmts.) de la vulva para prevenir la remoción por otras vacas curiosas.
10. Remueva esta inserción 7 días después, tirando suavemente de la punta del CIDR®

*Source: Genex Cooperative, Inc./Cooperative Resources International,  
Shawano, Wisconsin, USA*