### **Instructions for Application of ESTROTECT™:**

For maximum retention of ESTROTECT™, it is very important to use proper application procedures.

1. Warm the ESTROTECT™ patch before application. The adhesive backing is heat activated. Patches should be warmed to human body temperature. This can be achieved by placing inside a coat near the body, placing inside a container near a hot water bottle or setting the patch next to a heat register. The different warming possibilities are endless.
  2. Wipe and clean the area of the animal where the ESTROTECT™ patch will be applied. Dirt, dust, oil, shedding or loose hair and excess moisture must be removed before application. If the animal is bedded with sand, it is very important to thoroughly remove all sand from the tailhead.
  3. Apply the ESTROTECT™ across the animal's backbone half way between the hooks and pins. Rub the patch onto the hair with bare fingers. This rubbing will not damage the silver coating. Press down hard.
- Do not clip or shave the hair. If the hair is extremely long, do not trim it shorter than one inch.

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- *Source: Genex Cooperative, Inc./Cooperative Resources International, Shawano, Wisconsin, USA*